

*Symphony™*

**Brachytherapy  
Transfer Device**



Patent(s) Pending


Shown with leg extension removed



positioning  
patients for life.®

[www.Qfix.com](http://www.Qfix.com)

RT-5100-05 Symphony Brachytherapy Transfer Device

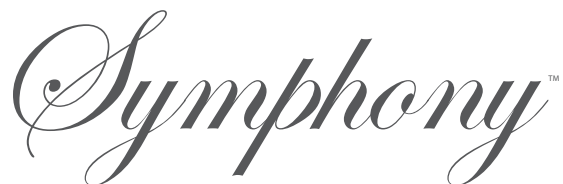

## Symphony Brachytherapy Transfer Device

With Qfix's new Symphony Brachytherapy Transfer Device, the Symphony Patient Transport System enables a wide variety of efficient workflow solutions for prostate and gynecological brachytherapy procedures.

This revolutionary, modular system is designed to maximize workflow efficiencies and throughput by seamlessly aiding in the setup, positioning, and transfer of the patient for procedures involving a range of imaging and radiation treatment modalities. All Symphony Transfer Devices utilize a low-friction air bearing to move patients from one modality to another, eliminating the need to manually lift the patient. The Symphony air bearing technology allows the patient to be transferred onto and off of pads and mattresses.

The Symphony Brachytherapy Transfer Device incorporates an easily removable leg extension and integrated side rails to support stirrups, steppers, and other rail-mounted accessories. Symphony provides workflow improvements for high dose rate and low dose rate brachytherapy procedures from patient setup and applicator or needle insertion through imaging and treatment.

### Patient Setup

Using the Symphony Brachytherapy Transfer Device, the patient can be set up on the Symphony Patient Trolley, a surgical table, or most any other patient-supporting bed or table.

### Insertion

Once sedated with the patient placed in the lithotomy position, the needle or applicator insertion is performed as usual.

### Imaging

The Brachytherapy Transfer Device is compatible with CT and MRI imaging, providing clinicians flexibility in approach for planning and placement confirmation. The unique design minimizes CT and MRI artifacting.

### Treatment

Patients may be treated on the transfer device directly on the trolley or transferred to another table or bed.

2008078\_EN\_A

This brochure contains information about products which may or may not be available in particular countries. Each country has specific laws and regulations governing the commercialization of medical devices and the communication of information regarding medical devices in printed and digital media. Products appearing in this brochure or on our website may or may not have received approval, clearance, or marketing authorization by a governmental regulatory body in any particular country, or may have received approval, clearance, or marketing authorization for different indications and restrictions in different countries, or may be for investigational use only. A product's appearance in this brochure should not be construed as a solicitation or promotion for said product, nor any indication which is not authorized by the laws and regulations of the country where the reader resides.



Qfix  
Avondale, PA  
+1 484.720.6053  
sales@Qfix.com  
www.Qfix.com